

PERFLOCK Tray

INSTRUCTION FOR USE

【Warning】

FOR DENTAL USE ONLY

Do not use in healthcare professionals and patients with a history of hypersensitivity to nickel or chromium, including rashes and dermatitis.

【Shape, Structure, Principle】

Shape

Hole type

Type	Size
Full Jaw Upper	XL, L, M, S, XS
Full Jaw Lower	

Material

Stainless steel

【Purpose or Effect of Use】

This instrument is used to place, hold, and press the impression material during impression taking.

【How to Use】

- (1) Sterilize before use.
- (2) Inject impression material into the tray.
- (3) Place the impression in the patient's mouth and take an impression according to the instructions for use of the impression material.
- (4) Wash and sterilize properly after use.

Precautions related to usage, etc.

The size of the tray to be applied, the amount of impression material to be placed, and the looseness of the impression should be appropriate, as it may cause a vomiting reflex in the patient. Also, the patient's condition should be carefully monitored while impressions are being taken.

【Caution】

- (1) Sterilize before use.
- (2) Always inspect the product before use.
- (3) Damage, wear, corrosion, deformation, dropout, or other damage or deterioration is confirmed, it shall not be used
- (4) Do not apply a strong shock to the product as it may cause breakage, etc.
- (5) Not to be used for any purpose other than those described in the [Purpose or Effect of Use] section.
- (6) Only qualified dental personnel shall use the equipment.
- (7) If itching, redness, rash, or other allergic symptoms occur, discontinue use, rinse thoroughly with water, and consult a physician.
- (8) Remove blood, body fluids, tissues, etc., as soon as possible after use, and wash, disinfect, and sterilize.
- (9) Be careful when this product is immersed in corrosive solutions such as EDTA solution, sodium hypochlorite, etc. for a long period of time, as there is a risk of corrosion, etc.
- (10) When disposing of the product, take care to prevent infection and properly dispose of it in accordance with the relevant laws and regulations and the guidance of each local government.

【Storage and Expiry】

- (1) Store in a clean location away from high temperatures, low temperatures, high humidity, direct sunlight, moisture (water leakage), exposure to corrosive agents and their vapors, and subject to pressure (physical load) and contamination.
- (2) When storing sterilized items, keep them in a clean place to prevent contamination.
- (3) Do not store with rusting equipment.
- (4) To prevent contact corrosion, do not allow metallic instruments made of different materials to come into contact with each other during storage.
- (5) Properly stored and controlled so that only dental personnel may touch them.

【Matters pertaining to Maintenance and Inspection】

- (1) When cleaning/disinfecting after use, rubber gloves, protective glasses, masks, etc. should be worn to prevent infection.
- (2) Blood, body fluids, tissues, etc. adhering to the equipment shall be removed securely by washing with running water or immersing in a cleaning solution, etc. before they dry and solidify.
- (3) When using cleaning/disinfecting agents, use anti corrosive cleaning/disinfecting agents and clean/disinfect according to the instruction manual for the cleaning/disinfecting agent to be used.
- (4) Avoid using metal scrubbers, polishing powder, etc., as they may scratch the surface of the instrument and cause corrosion.
- (5) After washing/disinfecting, rinse thoroughly with purified water and dry.
- (6) After washing, perform high pressure steam sterilization. For details, follow the instructions for the high pressure steam sterilizer. Example: 125°C for 10 minutes.
- (7) When using cleaning, disinfection, or high pressure steam sterilization equipment, do not allow the equipment to come into contact with each other.
- (8) Dry before storing.
- (9) Do not exceed 136°C during sterilization (including drying) as this may impair the quality of the product.

【MANUFACTURER INFORMATION】

**Premium Plus Japan Co., Ltd.**

5th Floor, 1-1-31, Nishimidorigaoka, Toyonakashi,

Osaka 560-0005, Japan

T: +81 6-6845-0066

info@premiumplus.jp

www.premiumplus.jp