

CLEARSOLE

INSTRUCTION FOR USE

【INDICATIONS】

- Medium consistency laboratory Vinylpolysiloxane for counter mould in composite techniques with a transparent flask. Particularly suitable for extensive work.
- Vinylpolysiloxanes may cause irritation or other allergic reactions.
- FOR DENTAL USE ONLY

【CONTENTS】

CLEARSOLE	
PJ9700	CLEARSOLE 2x50mL cartridges 6x Mixing tips
PJ9710	CLEARSOLE 4x50mL cartridges

【DESCRIPTION / USE】

Laboratory transparent addition silicone for moulding and injecting light curing composites.

【INSTRUCTION FOR USE】

Insert the cartridge into the manual dispenser. The notch "V" on the basis of the cartridge should be facing down.

Lock down the appropriate device of the dispenser. Remove the colored cap by rotating ¼ turn counterclockwise.

By acting on the lever of the dispenser, dispense a small amount of material until the two components come out evenly.

Delete the extruded material and mount the mixing tip in the reverse order to the removal of the cap.

【STORAGE AND EXPIRY】

Store in a dry place at a temperature of 5-27°C (41-80°F). Protect from heat. The materials are guaranteed until the expiration date when stored in their properly closed containers. The production lot and the expiration date are indicated on the packaging.

【PRECAUTIONS AND WARNINGS】

Avoid contact with latex gloves.

Do not swallow, avoid contact with eyes.

【TECHNICAL DATA】

Mixing ratio	1:1
Mixing time (23°C/73°F)	Automixing
Working time (23°C/73°F)	1'
Setting time (23°C/73°F)	12'
Hardness - Shore A	74 ± 2
Linear dimensional change	< 0.20%
Recovery from deformation	> 99.8%

Higher temperatures reduce the times, lower temperatures increase them.

【LIMITATION OF THE RESPONSIBILITY】

The indications related to the products, both in oral form and written, are founded on the knowledge and experience acquired in dental field. The professional users are kept however to follow the indications for the use and to verify the fitness of the product according to the type of application. The professional user is assumed all the responsibilities for the consequential risks from an incorrect use of the products. A possible defectiveness of the products limits the responsibility of the manufacturer to the value of the products used by the professional.

【MANUFACTURER INFORMATION】**Premium Plus Japan Co., Ltd.**

5th Floor, 1-1-31, Nishimidorigaoka, Toyonakashi,
Osaka 560-0005, Japan

T: +81 6-6845-0066

info.japan@premiumplus.jp

www.premiumplus.jp